


Können die Grenzwerte der Temperaturmessung variabel gesetzt werden?

The limit values of the individual temperature ranges for no fever, slightly elevated body temperature, elevated body temperature and high body temperature can be selected as desired.

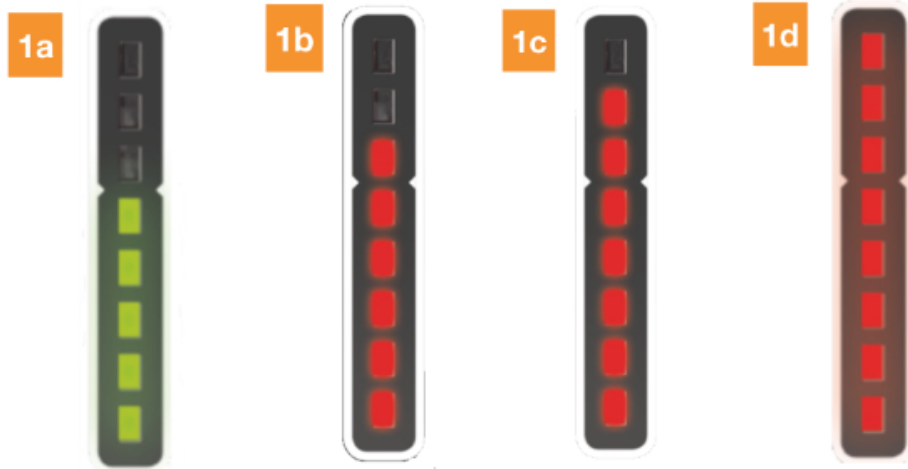

1. Temperature scale with 8x LED for temperature indication in four temperature ranges
 - a. LEDs 1-5 **GREEN**: Body temperature ok (36,5°C to 37,4°C)
 - b. LEDs 1-6 **RED**: Body temperature slightly increased (37.5°C to 38.0°C)
 - c. LEDs 1-7 **RED**: slight fever (38.1°C to 39°C)
 - d. LEDs 1-8 **RED**: high fever (from 39.1°C)

These settings are made in the SmartXcan web interface, see [Basic Functions SmartXcan](#)



Notice

Above 38 degrees Celsius, health authorities (e.g. the "US Center for Disease Control" US Department of Health) assume a fever.